

Diabetic Replacement Kit


for PneumaticWalker™ Diabetic System

(01R-M-MXW, 01R-L-LXW, 01R-XL)

The Diabetic Replacement Kit contains an insole and stocking for use with the PneumaticWalker Diabetic System. The kit size must correspond to the PneumaticWalker Diabetic System size, i.e. the Medium/Medium XWide kit is designed for the Medium and Medium XWide system, etc. The PneumaticWalker Diabetic System size can be easily identified by reading the I.D. label applied to the top outside edge of the PneumaticWalker.

All Aircast products are latex-free.

- 1** Remove front panel. Open foot flaps and toe cover.
- 2** Firmly grasp edge of pink insole and remove from brace. (Grey insole remains in brace.)
- 3** Remove backing from tape on bottom of new pink insole and firmly press on top of grey insole.

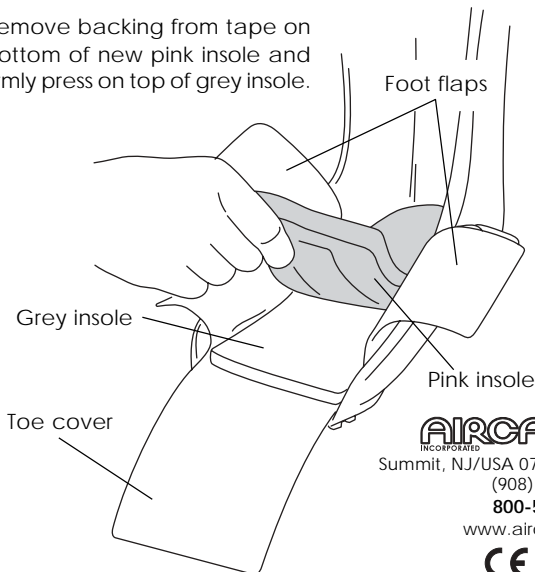

AIRCAST[®]
INCORPORATED

Summit, NJ/USA 07902-0709

(908) 273-6349

800-526-8785

www.aircast.com

CE
0123

R 11/6/01
01P140H
0136